— BEDIENUNGSANLEITUNG —

AUFSTEH ASSISTENT PATIENTENLIFT

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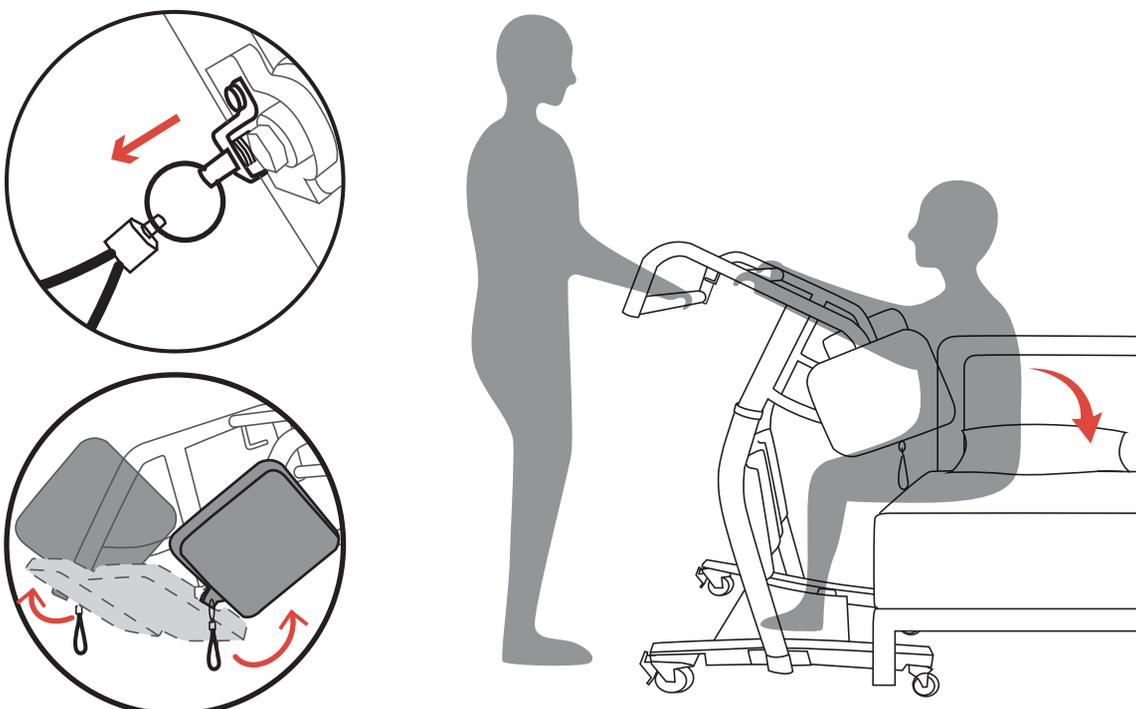
IV. ASSEMBLY INSTRUCTIONS

■ Notice

- Please read these instructions thoroughly before using.
- To avoid personal injury, please do not attempt to remove the stand assist from the box without assistance.
- Before using, ensure the product has been assembled according to the instructions in this manual.
- Scratches around the holes are due to the manufacturer's inspection process and do not indicate that the product has been used. This is a normal part of the quality control procedure.

1. Product Parameters

| Specification | Description |
|-----------------------|-----------------------------------|
| Model No | 6360-STA446BK |
| Material | Steel |
| Weight Capacity | 180 kg |
| Product Weight | 28 kg |
| Overall Dimensions | 86 cm(T) x 66 cm(B) x 118.5 cm(H) |
| Knee Pad Size | 29 cm x 18 cm |
| Seat Pad Size | 28 cm x 23 cm |
| Base Panel Size | 48 cm x 38 cm |
| Front Caster Diameter | 7.5 cm |
| Rear Caster Diameter | 10.2 cm |
| Package Quantity | 1 |

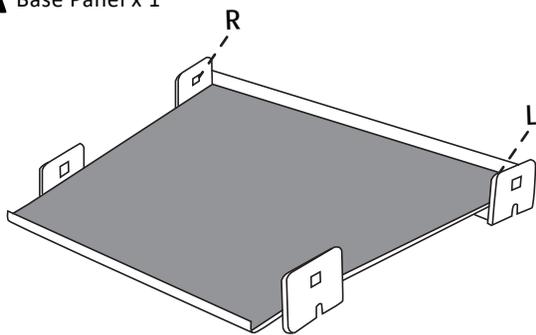
2. Part List

■ Notice

- Check the contents for completeness and make sure nothing is damaged.
- Remove all loose packing from the box.
- Carefully remove all the components from the box.
- All components have part numbers for identification.

Component


A Base Panel x 1


B-L Left Base x 1

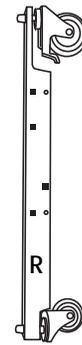
B-R Right Base x 1


C Knee Pad x 2

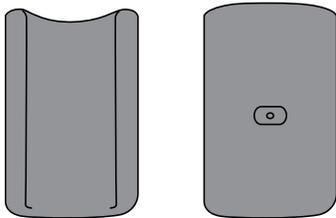
E-L Left Side Support Arm x 1

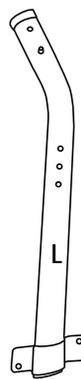
E-R Right Side Support Arm x 1

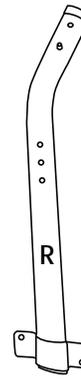
D Knee pads Support Unit x 1

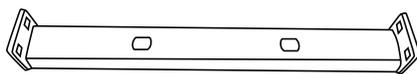
F Tubular Cross Bar x 2

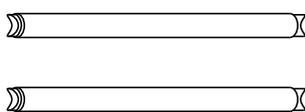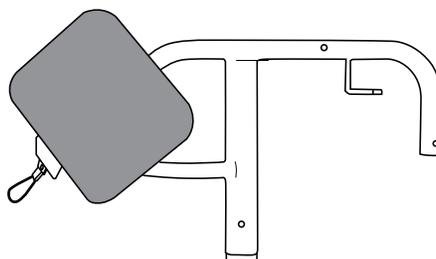G-L Left Seat Pad and Handlebar Support x 1

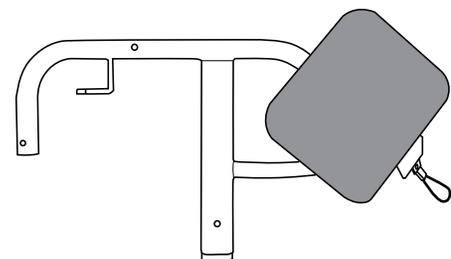G-R Right Seat Pad and HandleBar Support x 1

Fasteners and Tools

| | |
|---|---|
| <p>1</p> <div style="display: flex; justify-content: space-around; align-items: center;"> </div> <p style="font-size: small; text-align: center;">Long Carriage Bolt × 8 Flat Washer × 8 Acorn Nut × 8</p> | <p>2</p> <div style="display: flex; justify-content: space-around; align-items: center;"> </div> <p style="font-size: small; text-align: center;">Middle Carriage Bolt × 4 Flat Washer × 4 Acorn Nut × 4</p> |
| <p>3</p> <div style="display: flex; justify-content: space-around; align-items: center;"> </div> <p style="font-size: small; text-align: center;">Short Button Socket Head Cap Screw × 2 Flat Washer × 2</p> | <p>4</p> <div style="display: flex; justify-content: space-around; align-items: center;"> </div> <p style="font-size: small; text-align: center;">Short Carriage Bolt × 2 Flat Washer × 2 Acorn Nut × 2</p> |
| <p>5</p> <div style="display: flex; justify-content: space-around; align-items: center;"> </div> <p style="font-size: small; text-align: center;">Long Button Socket Head Cap Screw × 4 Hex key × 1 Wrench × 1</p> | |

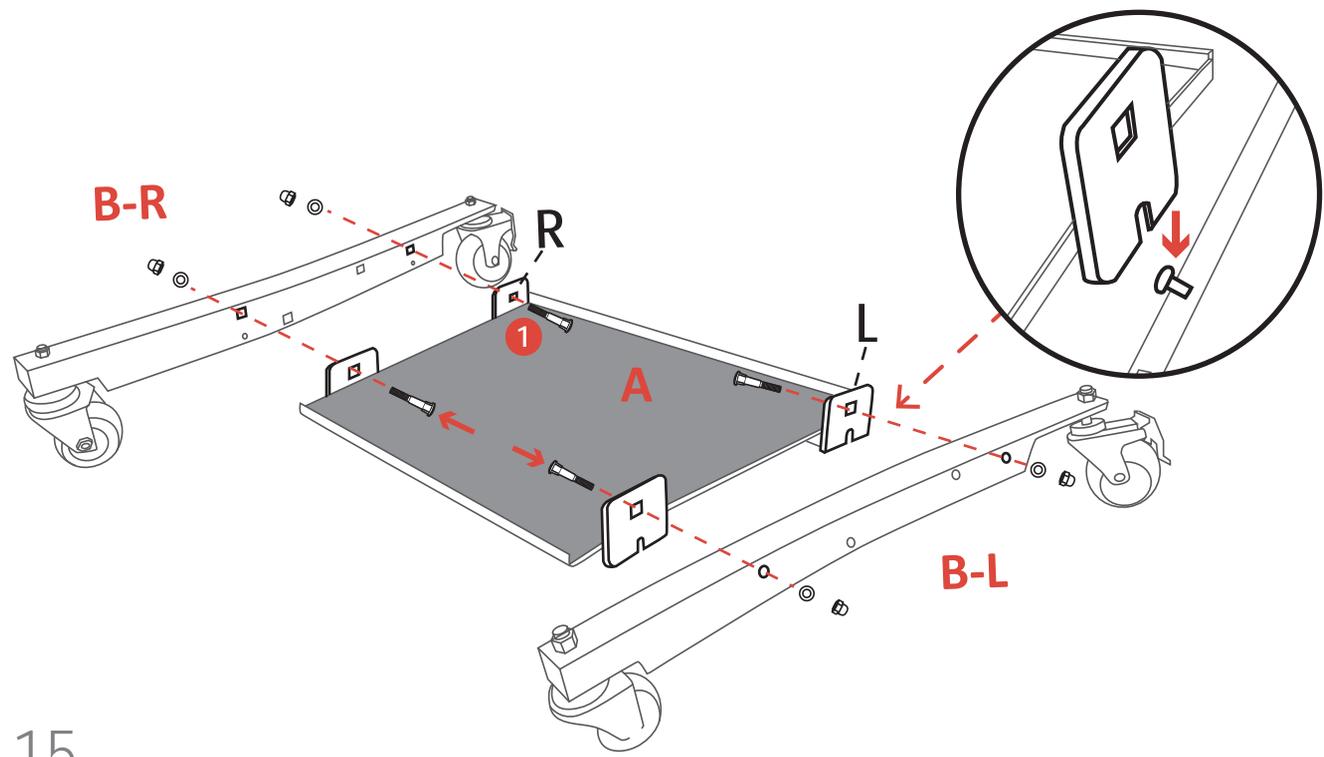
NOTE: Each fastener has a spare part in case of emergency.

3. Assembly Steps

Step 1 - Installation of the Base Unit

- Align the groove of the base plate with the white screws.
- Attach the left and right bases to the base plate using screw set #1.

Do not tighten the screws until step 6 is complete.


Step 2 - Installation of the Side Support Arms

- Align the holes of the side support arms with those on the base unit.
- Attach the left and right side support arms to the base using screw set #1.

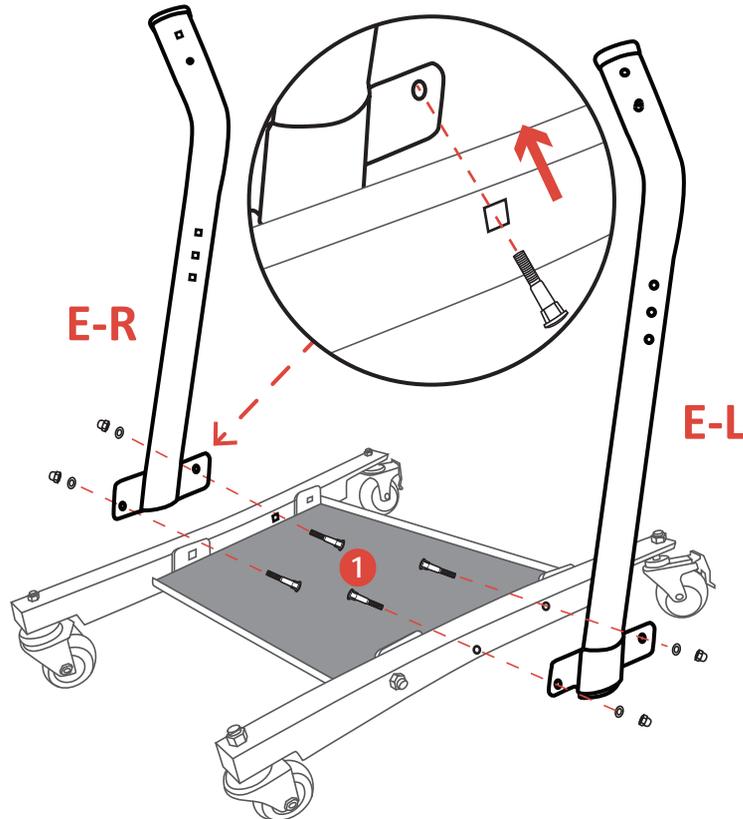
Do not tighten the screws until step 6 is complete.

Step 3 - Install Knee Pads & Support Unit

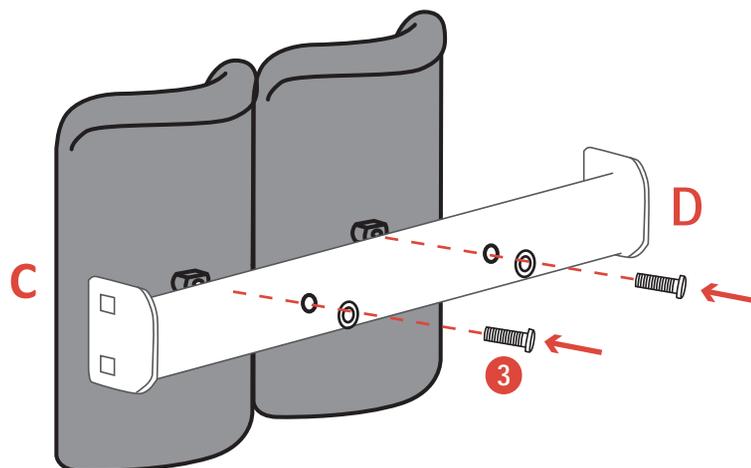- Attach the 2 knee pads to the support unit using screw sets #3 and then tighten the screws #3.


Step 4 - Install Assembled Knee Pads & Side Support Arms

- Attach the assembled knee pads to the two side support arms using screw set #2 and make sure that the square holes are facing inward.

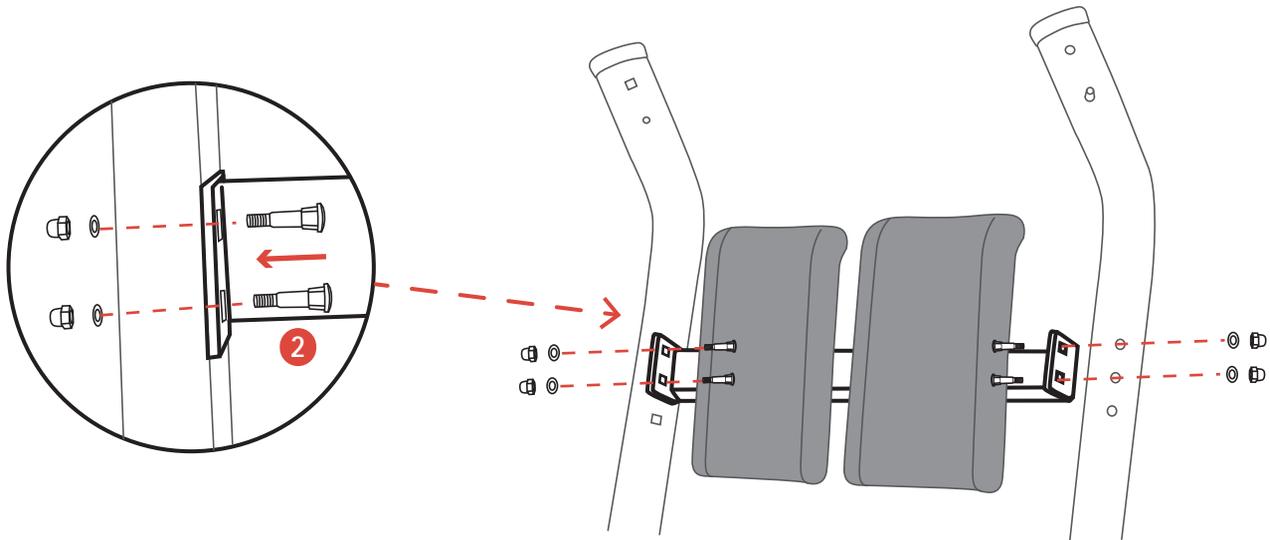Do not tighten the screws until step 6 is complete.

NOTE:

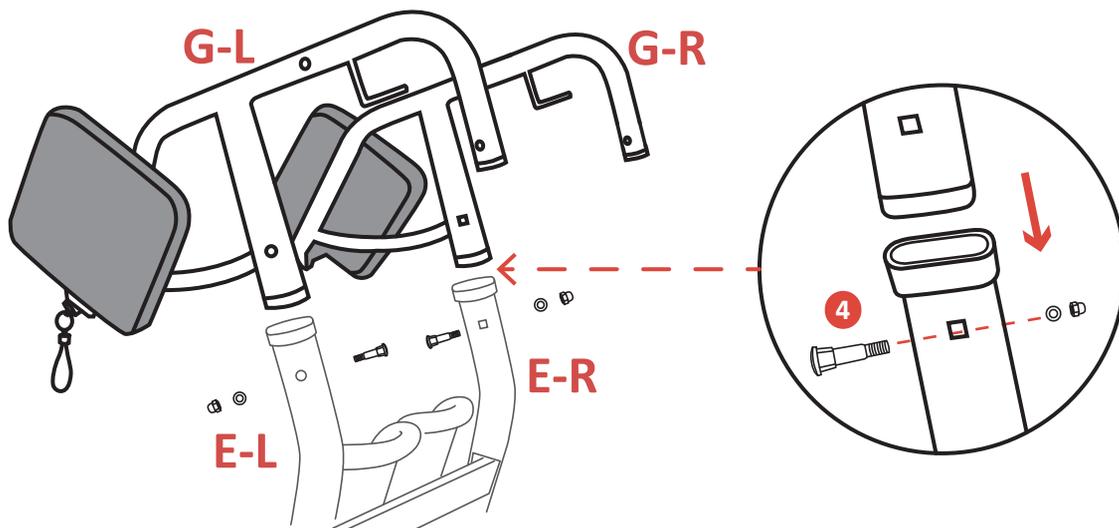
Adjust the height of knee pads according to the patient's height, keeping the knees in contact with the pads.

Step 5 - Install Handlebar Support

- Insert the handlebar supports into the tubes at the top of the Side Support Arms using screw set #4.
G-L matches E-L, and G-R matches E-R.


Do not tighten the screws until step 6 is complete.


Step 6 - Install Two Tubular Cross Bars

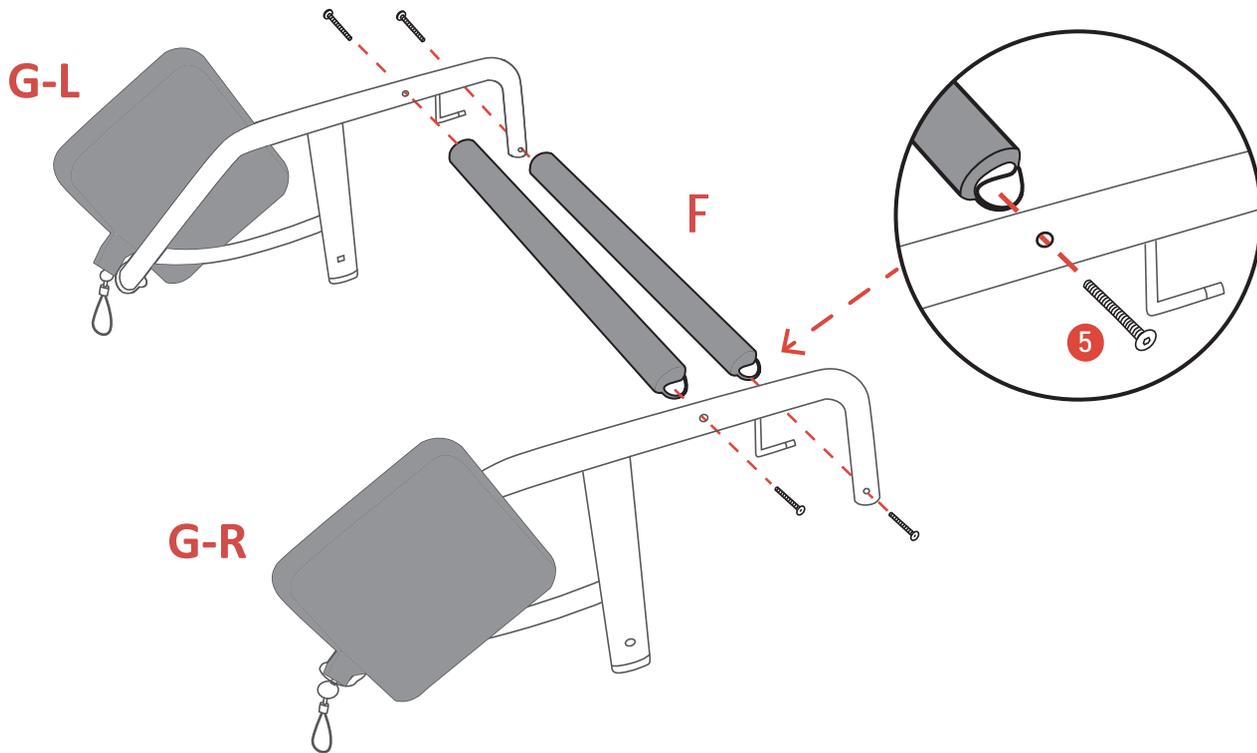- Align the holes of the tubular cross bars with the handlebar supports, and insert the screws from the outside using the provided hex key.

Product Overview

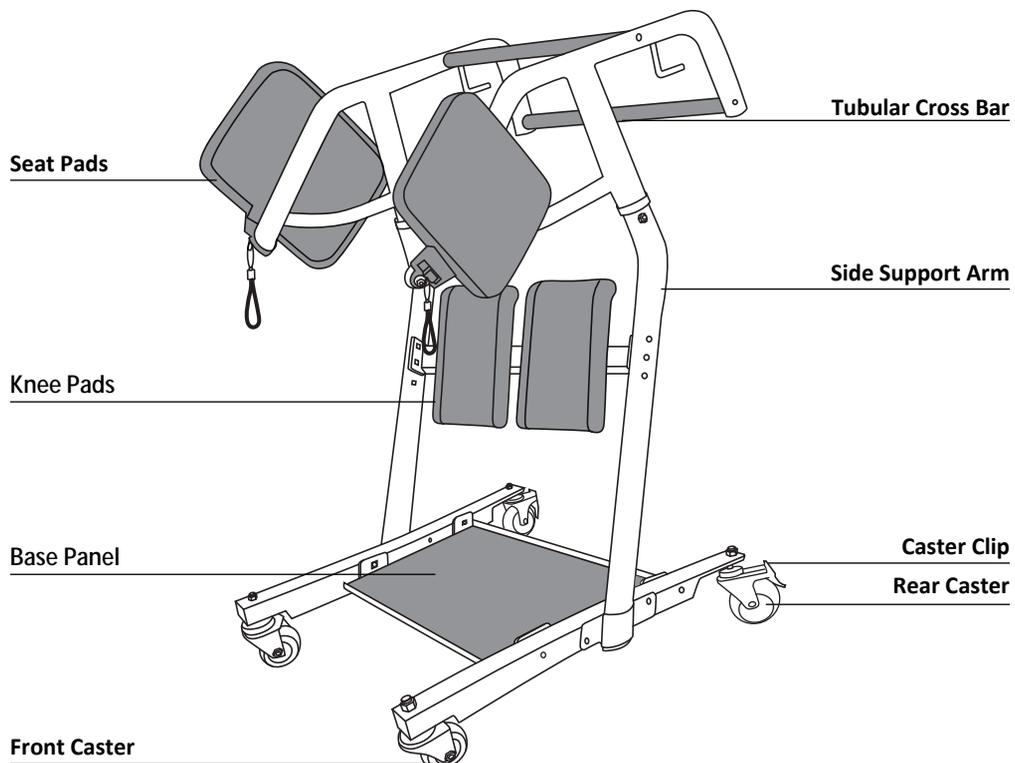V. OPERATING INSTRUCTIONS

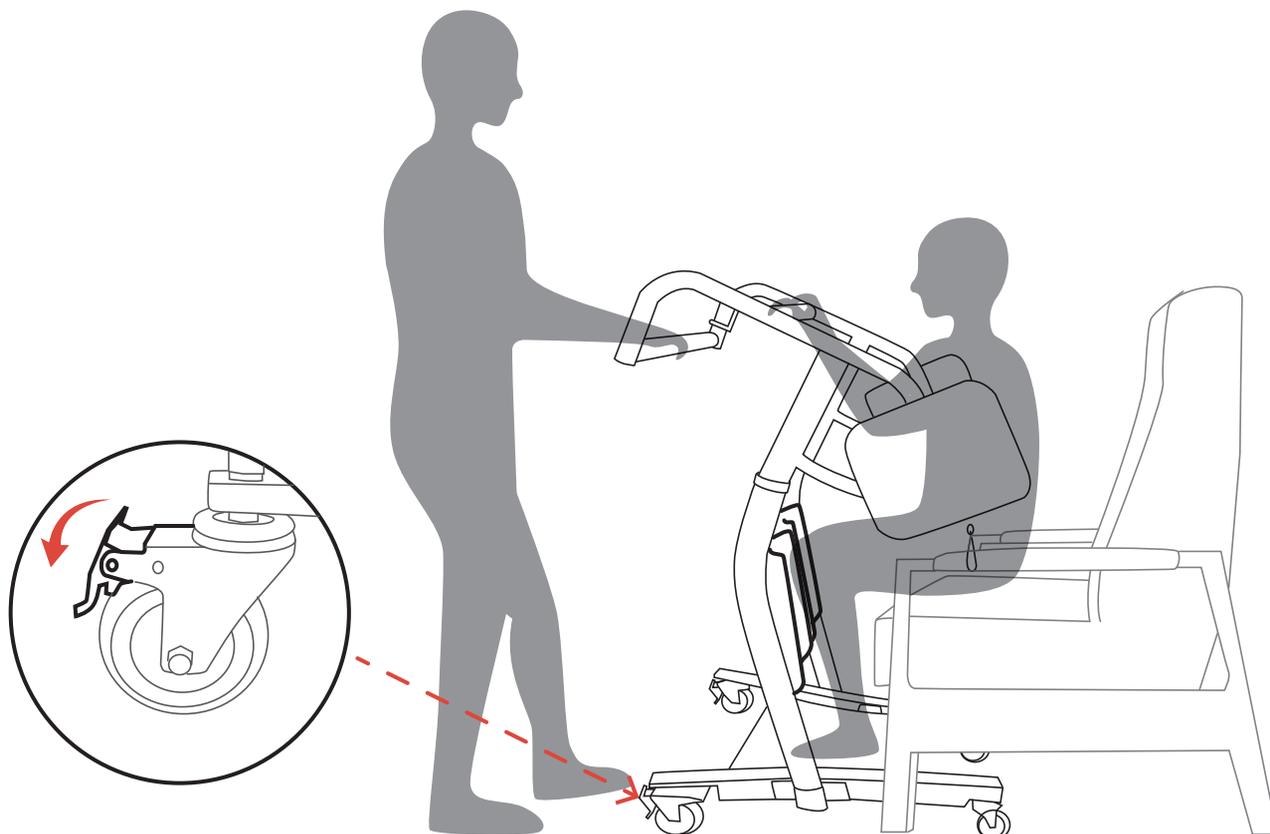
1. Operation Precautions

- Do not exceed the recommended weight limit of **180 kg**.
- Before use, make sure all assemblies are securely tightened and thoroughly read the operating instructions.
- The split seat units can be rotated upward for patient loading. Once the patient is in an upright standing position, the seat can be lowered to form a complete seat.

2. Operation Instructions


Step1 - Positioning the Patient

- Place the patient's feet on the base panel securely, and ensure that their knees and shins are properly aligned with the cupped knee pads.
- Lock caster clips of rear casters.



Step2 - Assisting the Patient to Stand

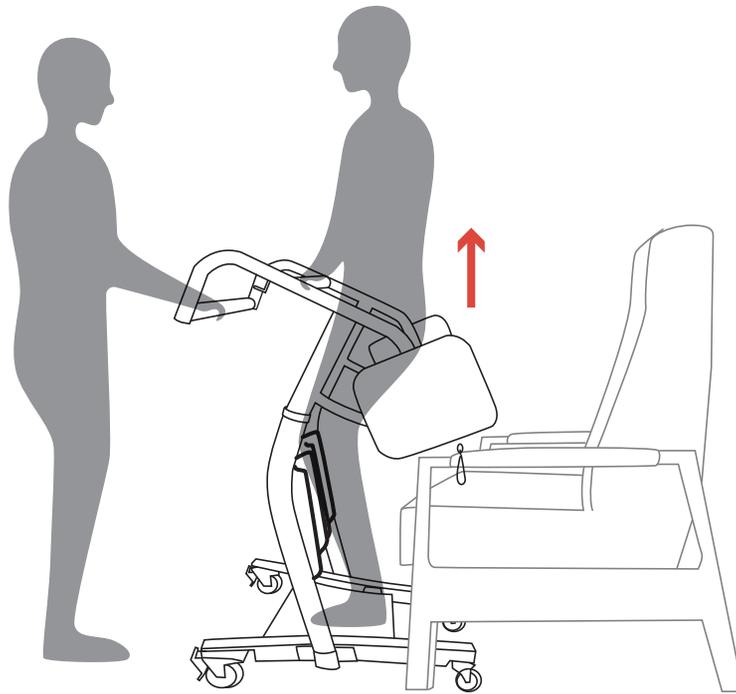- Guide the patient to hold the nearest cross bar, then use their strength to stand on the base panel.


Step3 - Assisting the Patient to Sit

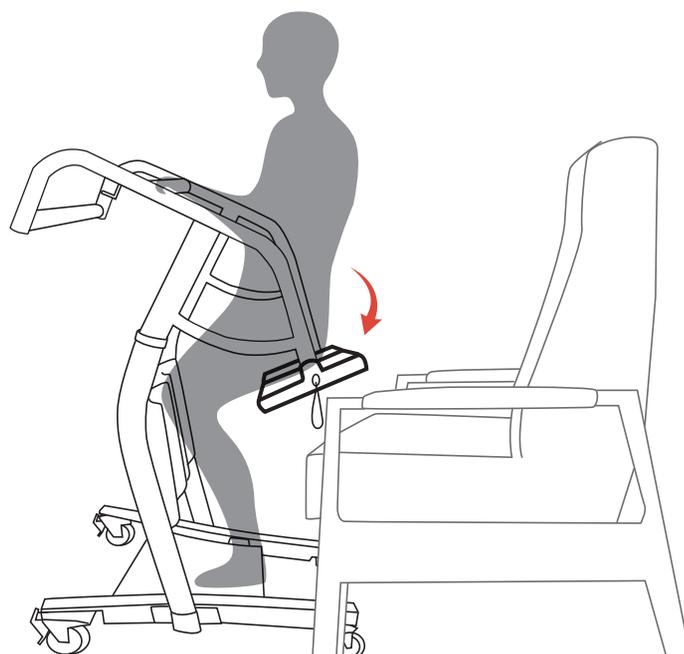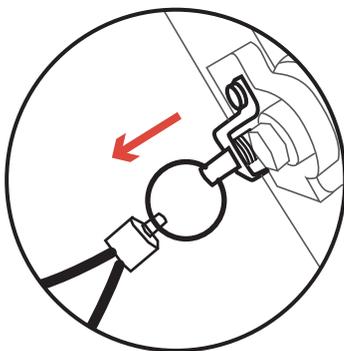- Pull the ring lock at the back of the seat and lower both split seat units to form a complete seat.
- Guide the patient to sit the complete seat, ensuring their knees and shins remain in contact with the pads. (The patient needs to keep holding the cross bar with both hands during this process.)


Step4 - Transferring to the New Surface

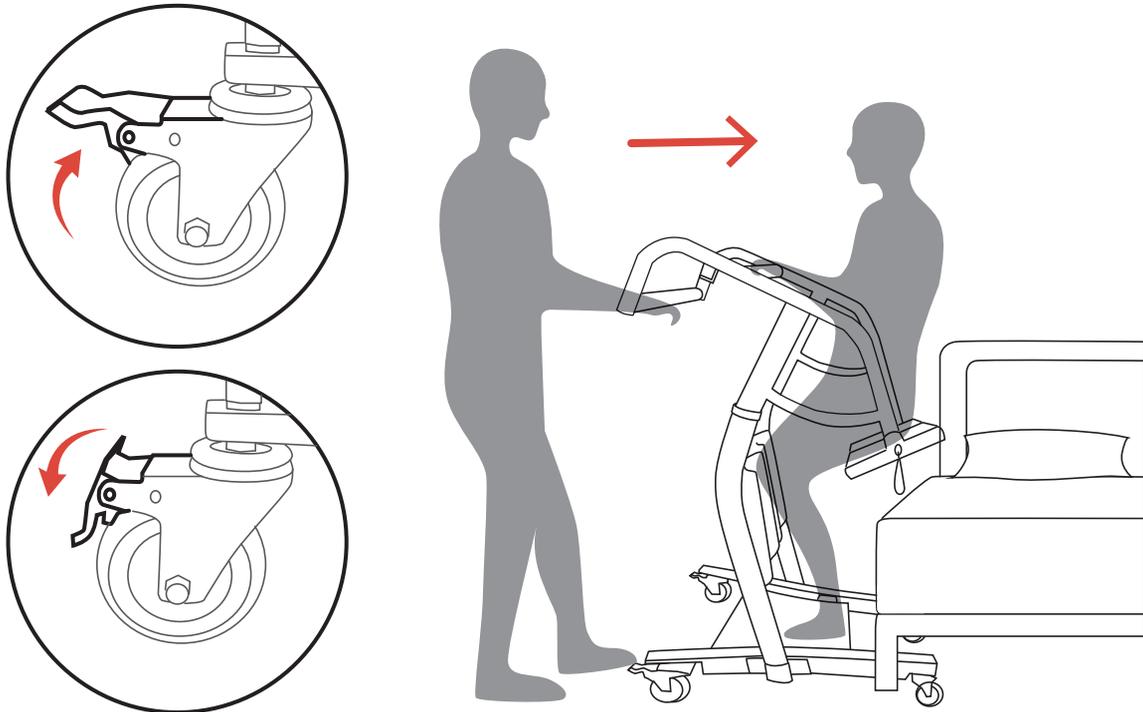- Unlock the rear casters, then transfer the patient.
- Move the stand assist to a suitable position of the new surface, then lock the casters in place.

Step5 - Complete the Transfer

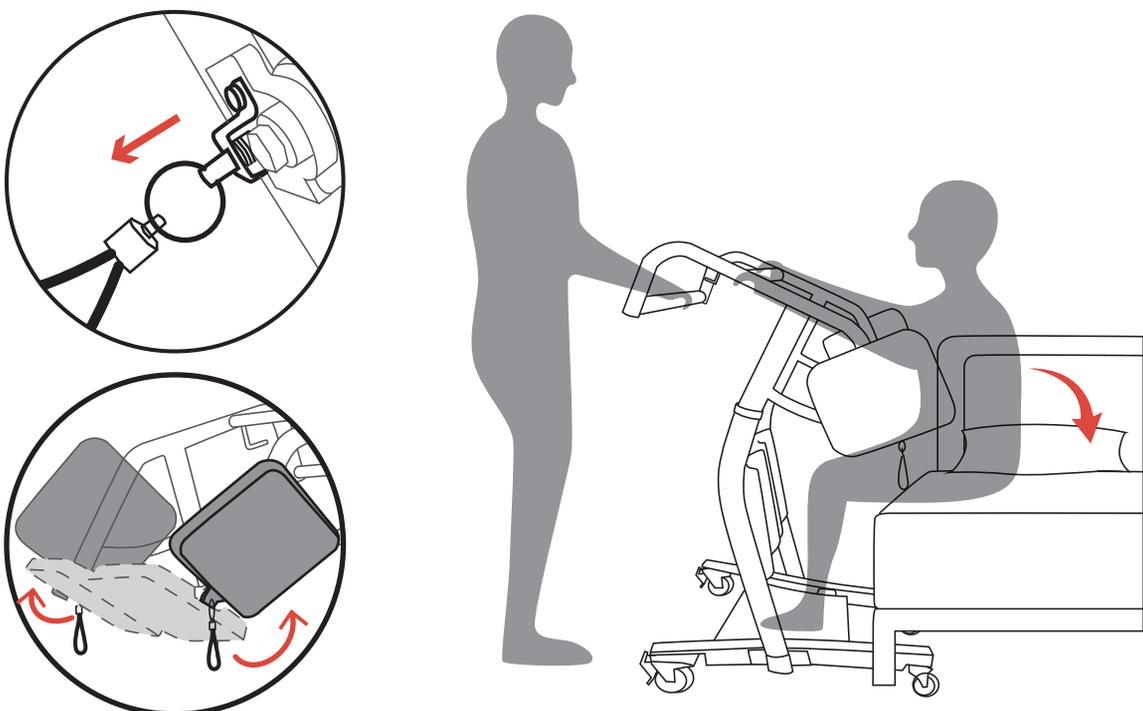- Pull the ring lock on the back of the seat to raise the two split seat units to their original parallel position beside the stand assist.
- Ensure the patient to sit on the new surface safely.
- Unlock the rear casters, and move the stand assist to its storage location.

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Manufacturer: Pfillo GmbH
Pfillo GmbH